



Cardiac Surgery Specialists

Luer Puller Catalog Number 410-125 Non-Sterile Patent Pending

Product Description

The Luer Puller is a stainless steel rod approximately 12 inches (30 cm) in length with a Male Luer that accepts a standard female luer connector. The device enables tubing with a female luer connector to be introduced into a sternal or thoracotomy incision and exit through a port or puncture incision.

Use of Device

1. Place male luer end through port or puncture incision until it is visible in the sternal or thoracotomy incision.
2. Attach female luer connector to luer puller. See Figure 1.
3. Withdraw Luer puller from port or puncture incision.

CLEANING

Disassemble and scrub the instruments and parts thoroughly using a soft brush and mild detergent. Remove all traces of blood and debris. Make sure all movement parts are cleaned thoroughly to prevent debris from interfering with movement. **It is recommended that the instruments and parts be Ultrasonically cleaned.**

STERILIZATION

The instruments may be steam or gas sterilized. Refer to the sterilizer manufacturer's instructions for correct time, temperature and pressure settings.

WARRANTY

ESTECH warrants that the instruments are free from defects in both materials and workmanship. Suitability for use of the instruments for any surgical procedure shall be determined by the user. ESTECH shall not be liable for incidental or consequential damages of any kind. The above warranties are in lieu of all other warranties either expressed or implied including any warranty of any merchantability or fitness for use.



**This device is non-sterile, sterilize before use.
Federal (USA) law restricts this device to sale by or on
the order of a physician or other licensed practitioner.**

SERVICE OR REPAIR /TECHNICAL INFORMATION

Contact information for service or repair or to request technical information:

ESTECH, Inc.

2603 Camino Ramon
Suite 100
San Ramon, CA 94583

Phone: 925-866-7111
Fax: 925-866-7117
Email: info@estechlics.com

European Authorized Representative:

Medical Partners BV
Groenendijk 39
Lage Zwaluwe
4926 RE Netherlands

Phone: +31.168.48 3732
Fax: +31.168.48 47 90



Figure 1
Luer Puller

Male Luer end accepts
female luer connector