



Cardiac Surgery Specialists

Instructions for Use
Ultra Instrument Holder Attachment
for USA II
Catalog Number 410-215
Non-Sterile
Patent Pending

Product Description

The Ultra Instrument Holder Attachment is reusable accessory for use with the 401-152T Universal Stabilizer Arms. When attached to the arms a 3-10mm axial instrument or scope can be held in position.

Use of Device

1. Check moving parts and function prior to use.
2. Attach Accessory Device to the 401-152T Universal Stabilizer Arm by sliding sleeve of the quick connection away from flexible arm and inserting Attachment Shaft into the connector with the detent lined up to the screw hole. *See Figure 1.*
3. Position the 401-152T Universal Stabilizer Arm on the retractor according to Instructions for use for provided with the device.
4. Insert Instrument or scope into clamp mechanism of Instrument Holder Accessory Attachment Tighten Thumbscrew control tight enough to hold instrument or scope while still allowing the instrument or scope to slide within clamp. *See Figure 2.*
5. Fix position of 401-152T Universal Stabilizer Arm according to Instructions for use provided with the device.
6. Position instrument or scope as desired and tighten thumbscrew until position of instrument or scope is fixed.

CAUTION

Attempting to move the position of the instrument or scope without loosening the thumbscrew and or stabilizer arm adjustment may damage the instrument or scope.

CLEANING

Disassemble and scrub the instruments and parts thoroughly using a soft brush and mild detergent. Remove all traces of blood and debris. Make sure all movement parts are cleaned thoroughly to prevent debris from interfering with movement. **It is recommended that the instruments and parts be ultrasonically cleaned.**

STERILIZATION

The instruments may be steam or gas sterilized. Refer to the sterilizer manufacturer's instructions for correct time, temperature and pressure settings.

WARRANTY

ESTECH warrants that the instruments are free from defects in both materials and workmanship. Suitability for use of the instruments for any surgical procedure shall be determined by the user. ESTECH shall not be liable for incidental or consequential damages of any kind. The above warranties

are in lieu of all other warranties either expressed or implied including any warranty of any merchantability or fitness for use.



This device is non-sterile, sterilize before use.
Federal (USA) law restricts this device to sale by or on the order of a physician or other licensed practitioner.

SERVICE OR REPAIR /TECHNICAL INFORMATION

Contact information for service or repair or to request technical information:

ESTECH, Inc.
2603 Camino Ramon
Suite 100
San Ramon, CA 94583

Phone: 925-866-7111
Fax: 925-866-7117
Email: info@estechlics.com

European Authorized Representative:
Medical Partners BV
Groenendijk 39
Lage Zwaluwe
4926 RE Netherlands

Phone: +31.168.48 3732
Fax: +31.168.48 47 90

